



GREEN TRACING STICKS:

We offer Green Tacking Sticks of diameter of 9.2mm x 113mm length, each weighing appx 10.5gms.

Properties & Advantages

- It is border moulding compound for accurate impressions.
- It is custom formulated for minimum shrinkage and accurate impressions ensured by good flow characteristics and setting time.
- It is suitable for muscle trimming work.
- Ideal for recording full depth of sulcus
- It may be used in conjunction with copper ring for individual crown impression.
- Make functional and relining impressions

Indications

- For border extensions on impression trays
Ideal for recording full depth of sulcus
Modify dentate tray by building up material where tooth has been extracted
Make functional and relining impressions

Composition

Thermoplastic Resin, Stearic Acid, Talcum, Colorant and excipients

Product Benefits

Fast and uniform plasticity
Easy to warm up
Quick hardening in cold water
Wide working area

Usage Instructions

- Soften the material over soft flame or by immersing in hot water (55-60 deg C)
- Place the softened, homogeneously kneaded mass in impression tray
- Apply thin layer of vaseline on surface to facilitate release of impression
- Quickly pass the surface of compound over a soft flame to obtain a sheen on it

· Insert the prepared tray in mouth with firm and even pressure, hold for 2 min

**· Trim the excess material & apply adhesive to prepare tray for final impression.
Thermoplastic material, protect from direct heat**

Packaging
10 sticks in individual box, Net wt: 90 gms (each stick 9 gms).